

# Disinfecting Shipping Containers Used For Redistribution



## **J82/E38: Blood Shipping Container and shipping components<sup>1</sup>**

Disinfect by using 70% isopropyl alcohol (typically 70/30 mix of alcohol to water) or other salt-based disinfectants. Spray and let dry before placing products into the container.

## **Credo EMT: Blood Shipping Container<sup>2</sup>**

**TIC coolant box:** Clean by using warm water and soap or alcohol. Decontaminate/clean by using an 70% isopropyl alcohol (typically 70/30 mix alcohol to water) or other salt-based disinfectants.

**Insulator (VIP):** Clean using a damp towel with soap or a rag with 70% isopropyl alcohol.

**Outer Nylon Bag:** Clean the outer bag by using a damp towel with a non-abrasive soap or rag with 70% isopropyl alcohol.



### Source:

1. <https://www.cdc.gov/infectioncontrol/guidelines/disinfection/disinfection-methods/chemical.html>

2. *Credo™ Pelican ProMed Series 4 8L User Guide*